
Otolift Line
User manual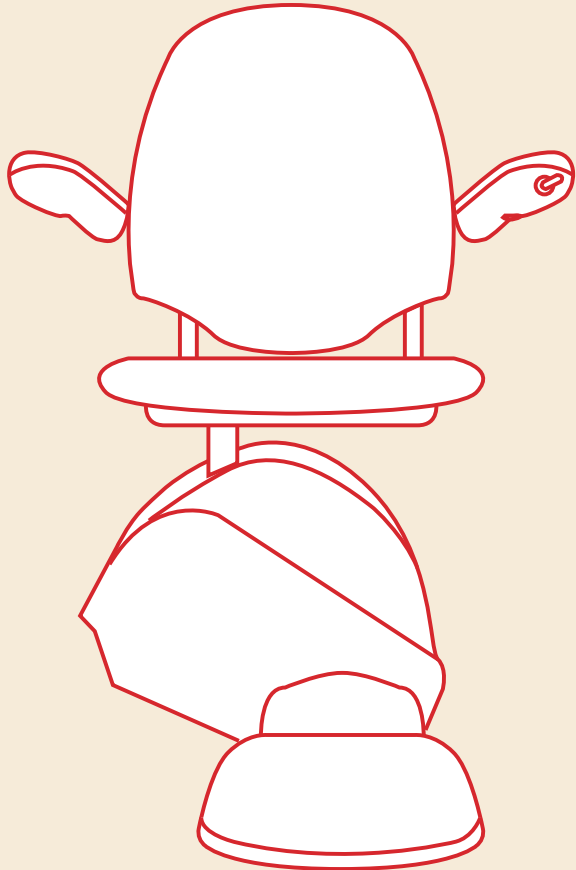

Index

| | |
|---------|------------|
| 4 - 5 | English |
| 6 - 7 | Nederlands |
| 8 - 9 | Français |
| 10 - 11 | Italiano |
| 12 - 13 | Español |

Please Note:

The intended use of this Stairlift is to carry one person only, at a maximum weight of 160kg up and down the stairs.

The Stairlift should not be used in the event of a fire.

The stairlift should be serviced at maximum intervals of 12 months, using only manufacturer approved parts and authorised personnel.

During normal operation, noise levels are not expected to exceed 70dBA

User guide

Otolift Line

1 Using the joystick

To make the lift move: push and hold the joystick in the direction you wish to travel. After a slight delay, the stairlift will begin to move until you release the joystick or reach the top/bottom of the rail.

2 Using your remotes

You can also use your remote controls to move your stairlift. Press the Up Arrow on the remote to move upwards and the Down Arrow to move the stairlift down the stairs.

3 Retractable seatbelt

Pull the seat belt buckle out and across the body then clip into the locking side with the red button. Once the lift has stopped and you are ready to get off, hold onto the belt and press the red button to release. The belt will then retract back out of the way.

Seatbelt must be worn at all times.

4 Swivel seat

Your stairlift comes equipped with a swivel lever as standard. Once you reach the top of the stairs pull the lever upwards. This will allow the seat to swivel out so you can easily step off. To swivel back lift the lever again and swivel back to the position, once you let go the lever will safely slot into position.

If you have an upgraded Powered Swivel. Push the joystick when arrived at the top to swivel. Either release the joystick to stop or wait for it to reach its limit. Don't worry the lift will not allow you to swivel too far. To swivel back push the joystick across to the downward direction.

5 Safety arm

For added safety the stairlift is fitted with a Safety Arm. The arm on the downside must be in its down position to be able to move

6 Safety edges

If your stairlift comes into contact with any objects on its travel, the stairlift will stop. The object must be removed to allow continuation of your journey. The lift will drive in the opposite direction of the obstacle to assist removal.

7 Arm rest controls

Located on the arm rest of the stairlift are: the Arm Display, Key Switch and the Holding Switch.

Arm Display - This display tells you in real-time what the stairlift is doing. Depending on current lift status the display will show the following:

LIFT READY - The lift is ready to use

MOVING UP - The lift is moving upwards

MOVING DOWN - The lift is moving downwards

ARRIVED TOP - The lift has arrived at the top

ARRIVED BOTTOM - The lift has arrived at the bottom

SEAT SWIVEL - The seat has fully swiveled

SERVICE - The stairlift is ready for a service

Key Switch* - The key must be inserted and turned to the ON position. To disable the stairlift turn to the OFF position. The Key can now also be removed for added security.

Holding Switch* - The switch turns your stairlift ON and OFF.

** Both the keyswitch & Holding Switch must be in the ON position for the stairlift to operate.*

8 Power cut?

Power ON/OFF Switch - This switch interrupts the battery supply. We recommend you leave it switched on unless you have a power cut. During a power cut switching off at this switch will maximise battery capacity for use.

In the event of a power cut or if the stairlift was to stop during travel, the stairlift can be wound down using the Hand winder tool provided.

On the main Chassis:

1. Turn OFF using the Power ON/OFF Switch

2. Remove the white plug

3. Insert the Hand Winder onto the shaft

4. Turn clockwise to move Up and anti-clockwise to move Down.

Troubleshooting Guide Otolift Line

Have a look at the **arm display**.
What does it say?

**LIFT
READY**

The lift is switched on and is ready to go.

**SWITCHED
OFF**

The lift is switched off.

1. Check that the **KeySwitch** is on.
2. Check that the **Power ON/OFF Switch** is on.

**OFF
CHARGE
AND
BEEPING**

The lift is not charging.

1. Check the **Charger** is switched on.
2. If **Hinge** model, make sure lift is parked.

There is no power to the lift.

1. Check that the **Charger** is switched on.
2. Check that the **Power ON/OFF Switch** is on.

**TILTED
ARM**

The downside arm is not lowered into position.

**HOLDING
SWITCH**

The **Holding Switch** is activated.
Check the switch on the arm.

**SEAT
SWIVEL**

The seat is in the swivelled position. Return the seat to the travel position.

**HINGE
OPEN**

The **Hinge** is open. The folding rail section is not in the down position. *Only on Hinge model.*

Try to run the lift in the **UP** direction:

**MOVING
UP**

The lift is moving up. The lift is operated by the Joystick or Remote Control.

**ARRIVED
TOP**

Lift has arrived at the top of the rail.

**UP
FOOTREST**

There is an obstruction on the upside foot-rest edge. Check footrest is clear.

**UP
CHASSIS**

There is an obstruction on the drive unit cover. Check that the cover is clear.

**MULTIPLE
CALLS**

There are multiple calls present, i.e. a **Remote Control** is pressed at the same time as the **Joystick**.

Try to run the lift in the **DOWN** direction:

**MOVING
DOWN**

The lift is moving down. The lift is operated by the Joystick or Remote Control.

**ARRIVED
BOTTOM**

Lift has arrived at the bottom of the rail.

**DOWN
FOOTREST**

There is an obstruction on the downside footrest edge or underpan. Check footrest is clear.

**DOWN
CHASSIS**

There is an obstruction on the drive unit cover. Check that the cover is clear.

**MULTIPLE
CALLS**

There are multiple calls present, i.e. a **Remote Control** is pressed at the same time as the **Joystick**.

Technician Required. Please call your local dealer:

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2860 AB Bergambacht